



2895 E. Blue Star Street
Anaheim, CA 92806

Products & Services for Dentistry

(714) 238-1131
FAX (714) 238-1161



Polyurethane Model and Die System

Shake well before using both Parts A and Part B

Spray impression or mold with release agent or silicone spray.

Mixing Instructions:

2 – Parts Activator (Part A) with

5 – Parts Filler (Part B)

Pour impression carefully to avoid any air bubbles then let it set.

Working Time: 3 – 3 1/2 minutes

Out of the impression in 45 minutes

Final Set time: 6 Hours

- Cups are reusable: After pouring the impression, let remaining material set in the cup then peel the material off.
- Please read MSDS before using this material.

www.etiempiredirect.com